



MOBILE COMPUTER CART
QUICK USER GUIDE

Have a Question? We're here to help.

Call us at 1-800-649-2278 or visit www.jacoinc.com/support

Exceptional service and support is vital to our customers, and Jaco understands that your EHR support needs to be timely, so your organization can keep the focus on providing exceptional care. We make it our priority to respond to all support requests within two business hours of receipt.

Technical Support

1-877-803-0101

Customer Service

1-800-649-2278

Jaco Hours

Monday - Friday
8:00 AM - 5:00 PM ET

Built with care, right here in America.

Jaco carts are designed for the rigors of the medical world. Made here in the USA, each Jaco cart is built from aircraft-grade aluminum. We cut it. We bend it. We weld it. We coat in beautiful, durable and antimicrobial finishes. The result is a flexible line of medical carts that enable high-technology care for every patient, in any medical environment, for many years to come.

Our adaptive manufacturing techniques and equipment give us the ability to build carts to your exact requirements. From multi-task EHR implementations to simply matching your brand's colors, every Jaco product can be custom-configured for your facility.

LCD Computer Cart Overview

Durable Aluminum Construction

Easily Adjustable Rotating and Tilting Monitor

Antimicrobial Powdercoat Finish


LED Charge Capacity Status; Push Button Controls; & Keyboard Light

Height Adjustment Lever

Optional Trac Steering
Green Lever: Engage Trac
Black Lever: Disengage Trac

Ergonomic, Retracting Keyboard/Mouse


Like steering a hospital bed

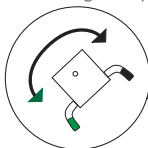

Disengage

Engage

Quiet Medical Grade Casters; 2 Locking

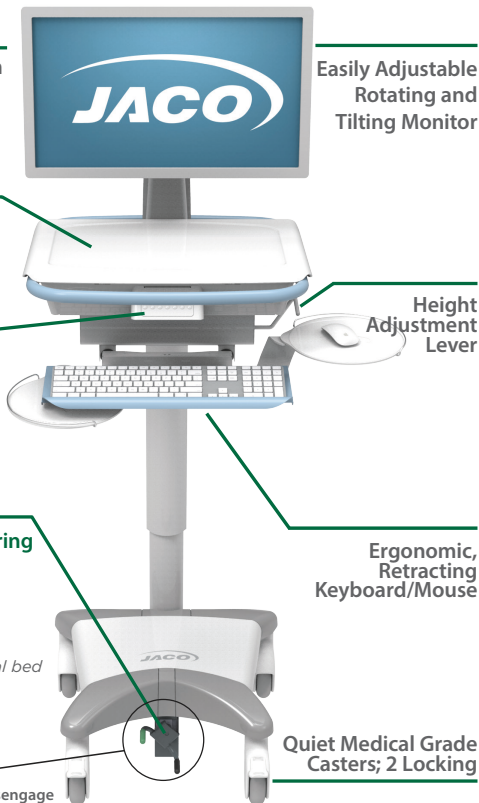

Laptop Cart Overview

Durable Aluminum Construction

Secure Laptop Inside Worksurface

Antimicrobial Powdercoat Finish


Optional LED Charge Capacity Status; Push Button Controls; & Keyboard Light

Height Adjustment Lever

Optional Trac Steering
Green Lever: Engage Trac
Black Lever: Disengage Trac


Ergonomic, Retracting Keyboard/Mouse

Like steering a hospital bed

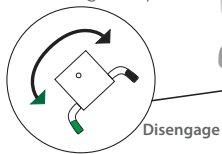


Engage

Quiet Medical Grade Casters; 2 Locking

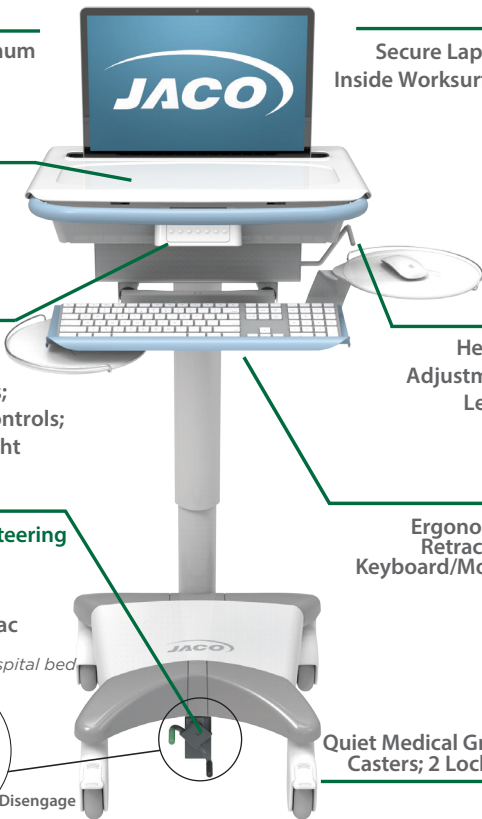

Tablet Cart Overview

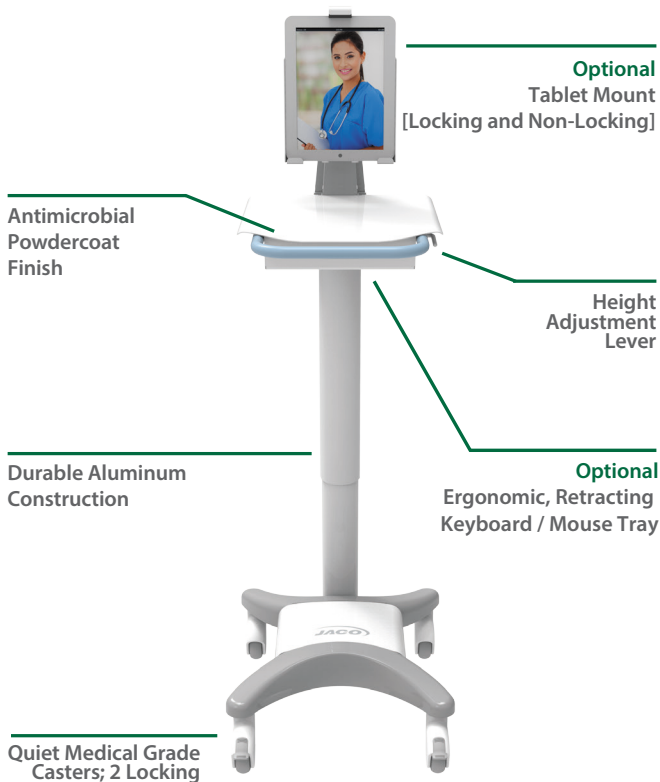

Power System Overview


LED Charge
Capacity Status;
Push Button
Controls; & Light

PowerBlade
Battery Release
Lever

Optional Onboard
Power System
[Available on UltraLite
500 Series]

PowerBlade
LiFe Hot Swap
Battery



Optional Hot Swap
Power System
[Available on
UltraLite 300 Series]

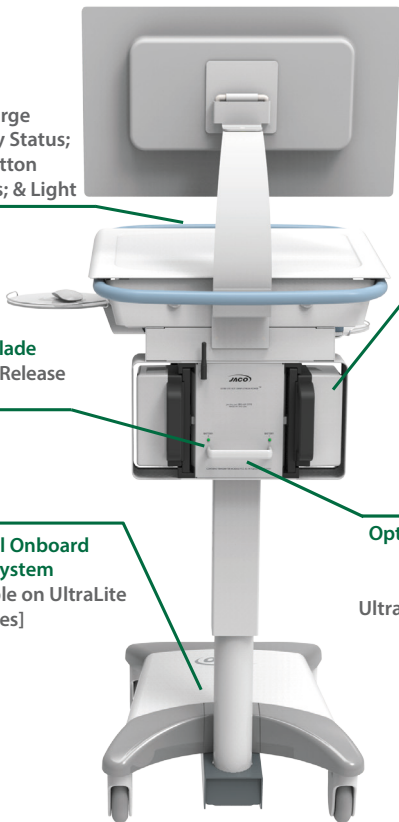

UltraLite 320 Model Shown

Storage Systems

Select from a wide variety of sizes and configurations to fit your needs. Available with drawers, bins, or trays. Locking, non-locking, and electronic locking options are all available.

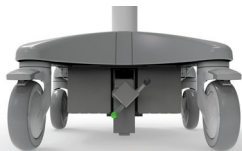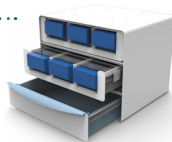

..... TRAC Steering Control

Engage and disengage a centered 5th wheel to improve straight line movement and cornering.

Visit www.jacoinc.com/trac for more information.


Keyboard and Mouse Adjustments

Slides in and out for easy use

Tilt Range: +15° to -5°

Mouse can be moved to left or right side

Optional Undermount Swing-out Mouse Tray

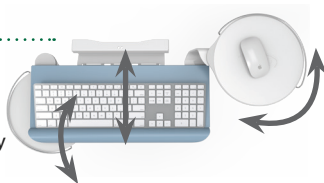

..... Nosecone Features

LED Battery Life Gauge

Keyboard Illumination

Optional Keypad Lock for Storage System

Monitor Adjustments

Pan Range: 180°

Tilt Range: 95°

Optional Vertical Range: 5"



Part Number: 25-0177

Revision: 02



Jaco Built in America



CERTIFIED BY

WOMEN'S BUSINESS ENTERPRISE
NATIONAL COUNCIL

JACO INCORPORATED

140 CONSTITUTION BLVD

FRANKLIN, MA 02038

WWW.JACOINC.COM

800-649-2278